
## RhinoPinch<sup>®</sup>Nasal Clip

## " An innovative NHS Scotland device to stop the flow of blood during nasal bleeds "



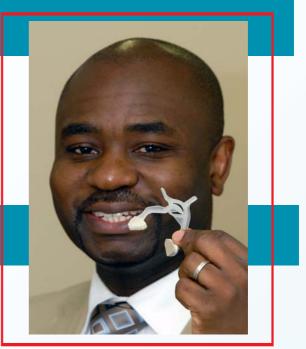

Dr Agbenyega - Clinical Teacher and Researcher NHS Inventor of the RhinoPinch® Nasal Clip

The RhinoPinch<sup>®</sup> Nasal Clip is a simple, single patient disposable device, individually wrapped in line with best practice infection control guidelines, fits all range of nasal sizes with non-latex nasal pads to ensure maximum protection and comfort

How it works Identify the spongy/fleshy tissue area (outside) just below the nasal/nose bone and by opening or spreading the padded area of the clip wide enough, apply pads to this area both inwards and upwards towards the face. Pinch the nose via the body of the clip from the padded end applying sufficient pressure until you are unable to breath through your nostrils. Maintain pressure via the



clip for between 12 to 15 minutes. The ratchet of the clip will retain the pressure to the nasal area. Repeat this procedure 3 times if bleeding continues.

## Benefits of the RhinoPinch<sup>®</sup> – Stops nose bleeds (Epistaxis) quickly and effectively

- Removes the risk of damage to the nostrils
- Meets infection control requirements
- Settles the patient's distress quickly
- Safe to use at any age
- Fits all ranges of nasal sizes

The RhinoPinch® evidence based clinically approved fit-for-purpose device developed within the NHS to deliver improved patient care in the management of nasal bleeds (Epistaxis).



Licensed to MDTi by Scottish Health Innovations Ltd

## EFFICIENT ~ EFFECTI



SCOTLAND

INVESTOR IN PEOPLE

ISO 13485

The Kace Building, Victoria Passage, Wolverhampton, West Midlands WV1 4LG, United Kingdom Company Reg No:04383577 Vat No:836 8531 02 www.mdti.co.uk email: info@mdti.co.uk Tel: +44 (0) 1902 778380 Fa: +44 (0) 1902 421360