

## HOOK-IT RANGE INSTRUCTIONS FOR USE



НООК-ІТ™ Ј-НООК

Blue 15mm (FKW010) and 21mm (FKW011)

Used to suspend IV and fluid bags from a wall bracket or 15mm/21mm width curtain track. J-HOOK will take up to two IV bag, max load 5kgs, (will break under human weight, antiligature).

The hook can be cleaned with soap and water or disinfectant wipe as recommended by the Trust Infection Control Policy. The J-HOOK can withstand multiple sterilisation cycles carried out at 135°.

When removing the J-HOOK from the curtain track please ensure J-Hook is LIFTED directly UP from track and NOT removed at an angle.

HOOK-IT<sup>™</sup> Wall Bracket

Make sure that the text on the front of the bracket is upright.

The wall bracket (FKW016) should be mounted approx. 5.5 ft on a wall from ground level. Position a bracket either side of a patient's bed if space allows and, in the bathroom/toilet.

The bracket can be cleaned with soap and water or disinfectant wipe as recommended by the Trust Infection Control Policy

For technical support and customer services please contact; Medical Devices Technology International Ltd (MDTi) Registered Office: The Kace Building, Victoria Passage, Wolverhampton, West Midlands WV1 4LG, United Kingdom Direct Line No. +44 (0) 1902 778380 Email : info@medti.co.uk Company No. 04383577 Vat No. 836853102 LRQ Approval Certificate No.4003482/A LRQ Approval Certificate No.4004298

MDTi commercialises products by the NHS for the NHS

