

MYAID™

Femmeze®

Femmeze®

Femmeze®

AN NHS INNOVATION

BPA and Latex free

Femmeze®

AN NHS INNOVATION

"An innovative device to improve Dignity and Self-management of Pelvic Organ Prolapse (Rectocele)"


A significant number of women suffer with the long-term and distressing condition commonly referred to as a pelvic organ prolapse (Rectocele). The condition occurs when the rectum herniates forward through into the vagina, obstructing the bowel from emptying normally.

The FEMMEZE® Vagina Trainer is a single user medical device that removes the need to use fingers and assists women to safely, simply and easily realign the rectum for passing stool.

The FEMMEZE® enables women who may have difficulty due to loss of articulation/dexterity to avoid discomfort and to maintain their independence and dignity.

The FEMMEZE® Vagina Trainer is an evidence based and dignified personal solution to help manage the condition in a hygienic and discrete manner.

Cleaning Instructions: This device is supplied as Non-Sterile, it is to be cleaned thoroughly all over with warm soapy water and then dried off before use or thoroughly wiped over with a sterile wipe. Before and after each use of the device ensure that you repeat the cleaning instructions. The device can also be cleaned in boiling hot water before use (allow the water to cool before removing). Do not use a disinfectant.

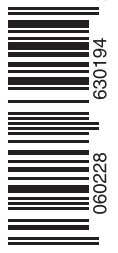

Indications of Use: The Femmeze Rectocele Device is for use as a manual, reusable and removable device placed into the vagina temporarily to train back or re-align the rectum tissue to treat a rectocele. The device is used to replace the use of pessaries or the fingers.

Product Code 40075



EU AR: MDTI Cyprus Ltd, 1 Kalyrmou Q MERITO, 4 floor, 6037 Lamaca, Agios Nikolaos, Kamareas, Cyprus.
Telephone No. +357 24343145 Email: info@mdti.eu

Manufactured by MDTI
Medical Devices Technology International Ltd
The Kace Building, Victoria Passage, Wolverhampton WV1 4LG, UK
Company Reg No 04383577
Tel: +44 (0) 1902 778380
www.mdti.co.uk info@mdti.co.uk



MYAID™

