

HOOK-IT® Range



INSTRUCTIONS FOR USE

A SIMPLE EFFECTIVE SOLUTION FOR MOUNTING DRIP/IV BAGS AN NHS INNOVATION



J-HOOK

Blue 15mm (FKW010) and 21mm (FKW011)
Used to suspend IV and fluid bags from a wall bracket or 15mm/21mm width curtain track.
J-HOOK will take up to two IV bags, max load 5kgs, (will break under human weight, anti-ligature).

The hook can be cleaned with soap and water or disinfectant wipe as recommended by the Trust Infection Control Policy. The J-HOOK can withstand multiple sterilization cycles carried out at 135°.

When removing the J-HOOK from the curtain track please ensure J-Hook is LIFTED directly UP from track and NOT removed at an angle.



Wall Bracket

Make sure that the text on the front of the bracket is upright.

The wall bracket (FKW016) should be mounted approx. 5.5 ft on a wall from ground level. Position a bracket either side of a patient's bed if space allows and, in the bathroom/toilet.

The bracket can be cleaned with soap and water or disinfectant wipe as recommended by the Trust Infection Control Policy

HOOK-IT® Range



Cleaning instructions

The product can be wiped clean with soap and water or alcohol. In addition, the product can be autoclaved for sterilization. Product can withstand multiple sterilization cycles carried out at 135°

Stress testing

Product can withstand loads of up to 12.5 newtons, equivalent to 12.5 litre fluid bags.

Life of Product

3 years



EU AR: MDTi Cyprus Ltd, 1 Kalymnou Q MERITO, 4 floor, 6037 Larnaca, Agios Nikolaos, Kamares, Cyprus mares, E-mail: info@mdti.eu



Medical Devices Technology International Ltd (MDTi)

Registered Office: The Kace Building, Victoria Passage, Wolverhampton,

West Midlands WV1 4LG, United Kingdom

Direct Line No. +44 (0) 1902 778380

Email : info@medti.co.uk

Company No. 04383577 Vat No. 836853102